

# BLOOD CELL COUNTERS - INSTRUCTIONS

## FEATURES/SPECIFICATIONS:

RIGHT AND LEFT RESET KNOB

BELL ALARM

NON-SLIP BOTTOM

ACID RESISTANT, HIGH-IMPACT PLASTIC BODY

## OPERATING INSTRUCTIONS:

- EACH OF THE KEYS ON THE COUNTER REPRESENTS A TYPE OF LEUKOCYTE, ACCORDING TO THE SCHILLING CLASSIFICATION.
- AS YOU OBSERVE AND IDENTIFY EACH TYPE OF LEUKOCYTE IN YOUR SAMPLE, PUSH DOWN THE KEY CORRESPONDING TO THE APPROPRIATE CELL TYPE
- PUSH FIRMLY TAKING CARE THAT YOU PUSH ONE KEY AT A TIME
- THE TOTAL COUNT DISPLAY KEEPS TRACK OF THE OVERALL NUMBER OF TIMES THE DIFFERENT KEYS HAVE BEEN PRESSED
- WHEN THE ADDITION OF ALL THE CELLS COUNTED REACHES 100, A BELL WILL SOUND. AT THIS POINT EACH INDIVIDUAL COUNT DISPLAY REPRESENTS THE PERCENTAGE OF THE CELLS IN THE SAMPLE.
- ONCE THE COUNT HAS BEEN COMPLETED, TURN THE KNOB AT THE END OF THE COUNTER UNTIL ALL WINDOWS ARE BACK TO ZERO.

## INSPECTION:

ENSURE THE UNIT IS KEPT CLEAN

PUSH DOWN ON EACH KEY TO ENSURE THE UNIT REGISTERS CORRECT NUMBER IN PROPER WORKING ORDER

TURN THE END KNOB, MAKING SURE UNIT CLEARS AND THE BELL RINGS IN PROPER WORKING ORDER. TO RESET THE COUNTER, ROTATE THE **TOTAL** FIELD TO ZERO THREE COMPLETE TIMES

## WARRANTY INFORMATION

STANDARD SIX MONTH WARRANTY AGAINST MANUFACTURING FAULTS

THE WARRANTY IS NOT VALID ON NORMAL WEAR AND TEAR, COSMETIC CHANGES OR DAMAGE CAUSED BY SOLVENTS AND CLEANING SOLUTIONS.